Owner's Guide

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
munchkin®

Flow Nipple Shield +

Téterelle Flow +

Pezonera Flow +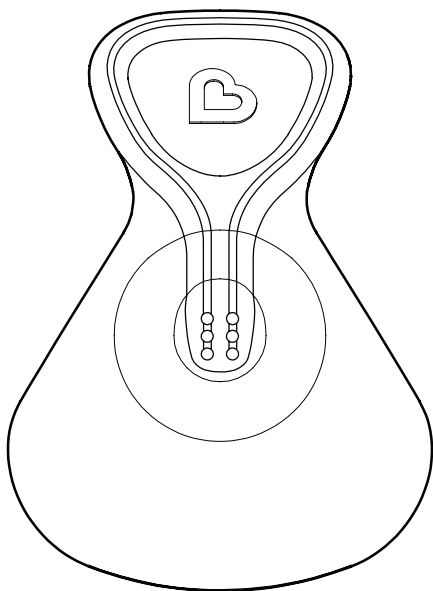

KEEP FOR FUTURE REFERENCE

Please read the instructions carefully before using this product.

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 IMPORTANT SAFEGUARDS	
Important Safeguards.....	5
 INCLUDED PARTS	
Included Parts.....	6
 HOW TO CLEAN	
Step 01 Cleaning.....	7
Syringe & Case.....	7
Flow Nipple Shield.....	7
 HOW TO DISINFECT	
Step 02 Disinfection.....	8
Option 1: Microwave.....	8
Option 2: Boil.....	8
 HOW TO USE	
How To Use.....	9
Storage.....	9
 CUSTOMER SERVICE	
Contact Us.....	10
Explanation Of Symbols.....	10
 RENSEIGNEMENTS IMPORTANTS	
Renseignements important.....	12
 PIÈCES INCLUSES	
Pièces incluses.....	13
 NETTOYAGE	
Étape 01 Nettoyage.....	14
Seringue et étui.....	14
Téterelle Flow.....	14
 STÉRILISATION	
Étape 02 stérilisation.....	15
Option 1 : Four à micro-ondes.....	15
Option 2 : ébullition.....	16
 MODE D'EMPLOI	
Mode d'emploi.....	16
Rangement.....	17
 SERVICE À LA CLIENTÈLE	
Nous Contacter.....	17
Explication Des Symboles.....	17

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ENGLISH
—
ANGLAIS
—
INGLÉS

INTENDED PURPOSE

Munchkin® Flow Nipple Shield is intended to support breastfeeding mothers by providing insights, assistance, and confidence to help them meet their breastfeeding goals. The shield supports the breastfeeding process by visually showing the flow of breast milk between a mother and her baby.

The shield is intended to cover and protect the nipples of a mother to support breastfeeding. It may also be used when a mother has flat or inverted nipples, sore nipples, or suffers from nipple pain.

NOTE: The Flow Nipple Shield is recommended for use only after your body is done producing colostrum. Typically, this may start from 2-5 days after birth.

READ ALL INSTRUCTIONS BEFORE USE. KEEP FOR FUTURE REFERENCE.

SAFETY INFORMATION

1. Use the product only for its intended purpose.
2. Do not use the product if the packaging has been opened or damaged.
3. Always follow cleaning, disinfecting, and storage instructions for proper care and use. Exposure to lint, dust, and sunlight may interfere with product performance.
4. Inspect the shield before each use. If you detect any damage or weakness, discard it immediately. Examples include unacceptable deterioration such as pitting, cracked seals, reduced flow, block cavity, particulate on the channel, separation of the channel from body and other abnormalities.
5. Flow Nipple Shield is intended for use by one mother. If feeding more than one baby, clean between uses.
6. Keep the shield away from children when not in use.
7. Use only Munchkin Cleaning Syringe and Sterilization Case for cleaning and storage.
8. If milk does not appear in the tube or you experience pain while breastfeeding or other issues, consult with a lactation consultant or physician.

QUICK START


Scan the QR code to learn more about Flow Nipple Shield:

1. Getting Started
2. How to Clean
3. How to Disinfect
4. FAQs
5. Flow Nipple Shield Line of Products & Replacement Parts

INCLUDED PARTS


Please check that all parts are included before you begin. If any parts are missing or damaged, please contact Customer Service at **800-344-BABY (2229)** or cserv@munchkin.com

Starter Kit comes with 3 parts: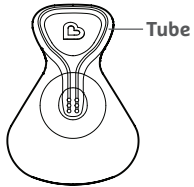


Small (16mm) or Medium (20mm) or Large (24mm) or XL (28mm)

A. SHIELD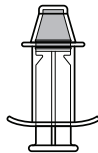

Small (16mm) or Medium/Large (20mm/24mm) or XL (28mm)

B. CLEANING SYRINGE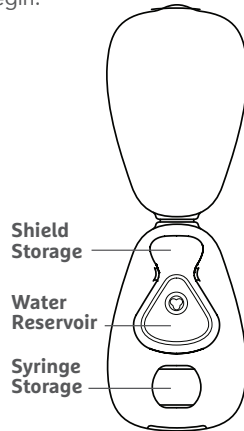


Shield Storage
Water Reservoir
Syringe Storage

Starter Case


C. STERILIZATION CASE

Replacement Kit comes with two parts: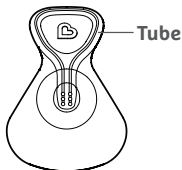

Small (16mm) or Medium (20mm) or Large (24mm) or XL (28mm)

A. SHIELD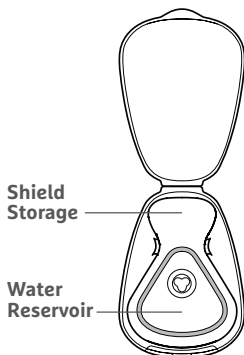

Shield Storage
Water Reservoir

Travel Case


B. STERILIZATION CASE

STEP 01: CLEANING

IMPORTANT: Follow the cleaning instructions for first-time use and clean the syringe and case as needed for subsequent uses.

SYRINGE & CASE

1. Wash your hands thoroughly.
2. Separate all syringe parts and case.
3. Wash all syringe parts and case in warm soapy water.
4. Leave to dry on clean surface.
5. Once dry, assemble all syringe parts and ensure a secure connection before proceeding.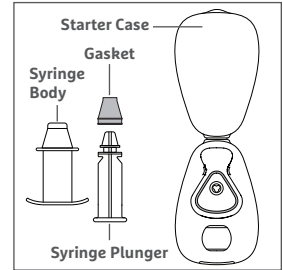

FLOW NIPPLE SHIELD

1. Prepare two small bowls:
 - A. Clean warm water mixed with a few drops of dishwashing liquid.
 - B. Clean drinking quality water.
2. Fill the syringe up to full capacity with (A) water and dishwashing liquid.




IMPORTANT: Make sure you are using the right size shield with the right size cleaning syringe.

3. Place the syringe inside the shield, making sure the syringe tip fits snugly in the nipple, and slowly inject all the water solution from the syringe into the nipple. It's normal to have some space below the shield and syringe.

TIP: Press down the tip of the nipple shield to help it suction better to the syringe tip before pushing down the syringe.

4. Be sure to push down the syringe slowly (count for 5 seconds) to ensure the milk tube is cleaned thoroughly. Ensure that the liquid solution is flowing through the tube and not squirting on the side of the syringe. Rinse the soapy syringe.
5. Repeat the same process with (B) clean water until no soapy water remains in the tube, then rinse the exterior.
6. Get rid of any remaining water left in tube by pushing air in with the syringe. Repeat as necessary.

7. Leave all parts on a clean surface and cleaning syringe in open storage case to dry. Place nipple shield back in storage case as silicone parts can attract dust and lint.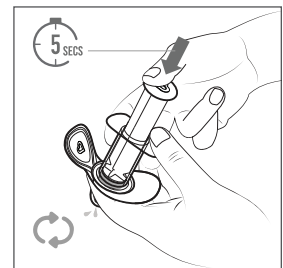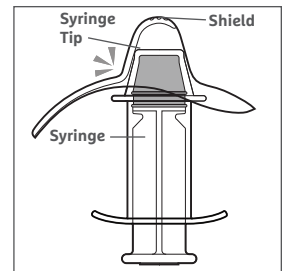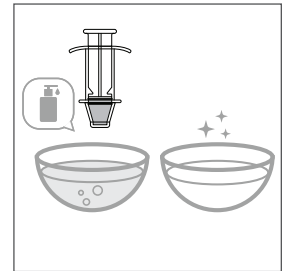

NOTE: If your breastmilk contains higher fat content, you may need to clean more than once to clear the channel completely.

STEP 02: DISINFECTION

IMPORTANT:

- Clean and disinfect nipple shield before first time use and at least once a day. There is no need to disinfect the syringe.
- To disinfect, choose to Microwave OR Boil and follow the instructions below.

CAUTION: Follow instructions carefully, as the case contains hot steam and water and may cause burns when removed from microwave if not handled properly. Add water to fill line to properly disinfect and avoid damage. Do not place the case on a metallic grill or dish when placed in the microwave. Discard and replace at first sign of weakness or damage.

OPTION 1: MICROWAVE

NOTE: Use the case according to the instructions below.

1. Clean nipple shield as described under

STEP 01 : Cleaning

2. Add 15ml of drinking quality water to the case up to the fill line.
3. Place shield inside the case then close the lid.
4. Place in microwave and heat based on chart below:





Microwave times	700 - 1000 W - 3 mins
	+ 1100W - 2 mins

5. Let it cool down for 10 minutes.
6. Drain water from case through openings by the hinge.
7. Get rid of remaining the water left in tube by pushing air in with the syringe. Repeat as necessary.
8. Leave on a clean surface or in open storage case to dry.
9. Place nipple shield back in storage case as silicone parts can attract dust and lint.

OPTION 2: BOIL

1. Clean nipple shield as described under **STEP 01: Cleaning**
2. Fill a pot with boiling water and bring to a boil.

IMPORTANT: Ensure water is boiling, let heat run an extra 2 minutes once steam is observed to ensure water reaches optimum boiling point before going to the next step. Be sure that there is ample water in the pot, so that no parts of the shield touch the bottom or the sides of the pot, as this could cause them to be damaged.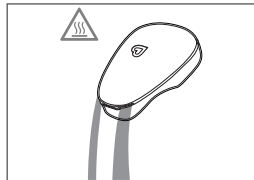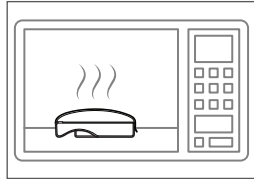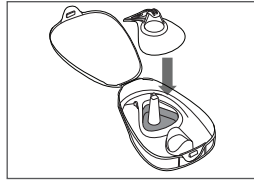
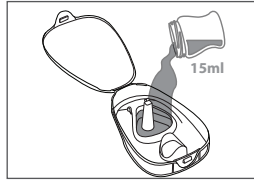


3. Carefully drop nipple shield into the boiling water and boil for 5 minutes.
4. Remove from boiling water. Let it cool down for 10 minutes.
5. Get rid of remaining the water left in Flow Nipple Shield tube by pushing air in with the syringe. Repeat as necessary.
6. Leave on a clean surface or in open storage case to dry.
7. Place nipple shield back in the case as silicone parts can attract dust and lint.

HOW TO USE


!/! WARNING : Wash your hands thoroughly with clean water and soap before touching the nipple shield and your breast. Dry your hands with a clean towel or single-use paper towel.

1. Place shield over the center of your nipple, with the milk channel facing upward towards the center for easy viewing. From your breastfeeding position, you should be able to see the tube.
2. Have your baby latch to your breast with the shield attached. The tube should be positioned between the baby's lips.
3. The milk should begin flowing through the tube but may take up to 90 seconds for letdown to happen. You will be able to observe the milk flow rate, its color and your baby's unique nursing patterns.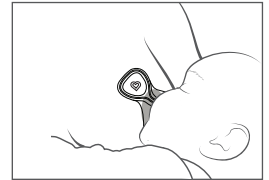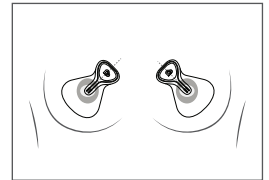

NOTE: Keep in mind that the milk flows through the tube as long as your baby is nursing. The consistency of your baby's suckling can vary and occasionally the baby may take a break, which will be observed as pauses or backward milk flow in the loop. The presence of air bubbles in the milk tube is normal.

4. Once feeding is done, move to cleaning.

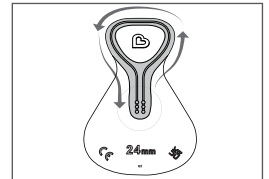STORAGE

- Place the clean, dry nipple shield and syringe in a clean storage case.
- Ensure that any remaining water or moisture is dried.
- Do not store nipple shield and syringe in a sealed container or bag as it may cause bacteria growth if not properly air dried.
- Avoid exposing to direct sunlight and avoid keeping in an enclosed car for prolonged period of time.

CONTACT US

If you have any questions about using this product, or if you wish to provide comments about this product, feel free to contact Customer Service at **800-344-BABY (2229)** or **cserv@munchkin.com**. Please visit **Munchkin.com** to check out our Travel Case, Replacement Nipple Shields, or Cleaning Syringes!

EXPLANATION OF SYMBOLS	
Symbols:	Meaning of symbols
	Medical Device Manufacturer.

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MANUFACTURED FOR ©2024 MUNCHKIN, INC, A WHY BRANDS COMPANY. 7835 GLORIA AVE., LOS ANGELES, CA 91406. MUNCHKIN BABY CANADA LTD., 3-8460 MOUNT PLEASANT WAY, MILTON ON L9T 8W7. WWW.MUNCHKIN.COM/PATENTS